


2Net Communicator


- 1 Unpack the enclosed the 2Net Communicator.

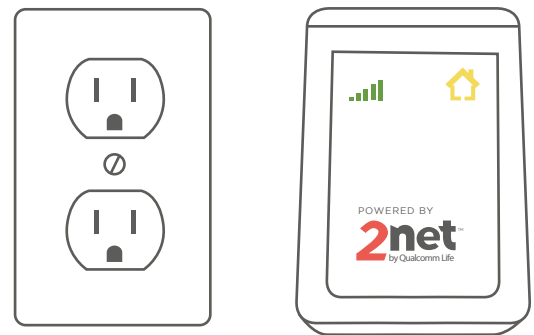

- 2 Plug the 2Net Communicator into the power outlet. Once it initiates, the 2Net Communicator will show the **green bars** and **yellow house** as seen below.



Communicator Lights

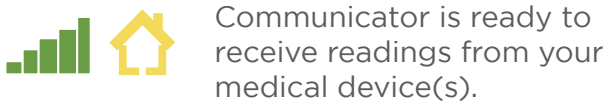

Communicator is ready to receive readings from your medical device(s).



Communicator is receiving readings from your medical device(s).



Communicator is sending readings from your medical device(s).



Communicator is resetting. If this is continual, please call the Vitals Monitoring Team at **1-888-875-0976** and then press "1".



Communicator is transmitting your device reading(s).