


Meet Your New Device

Bodytrace BP Monitoring System

Bodytrace BP was created to provide you comfort, reassurance and peace of mind by allowing you to check your blood pressure in the comfort of your own home.

Get to Know Your System

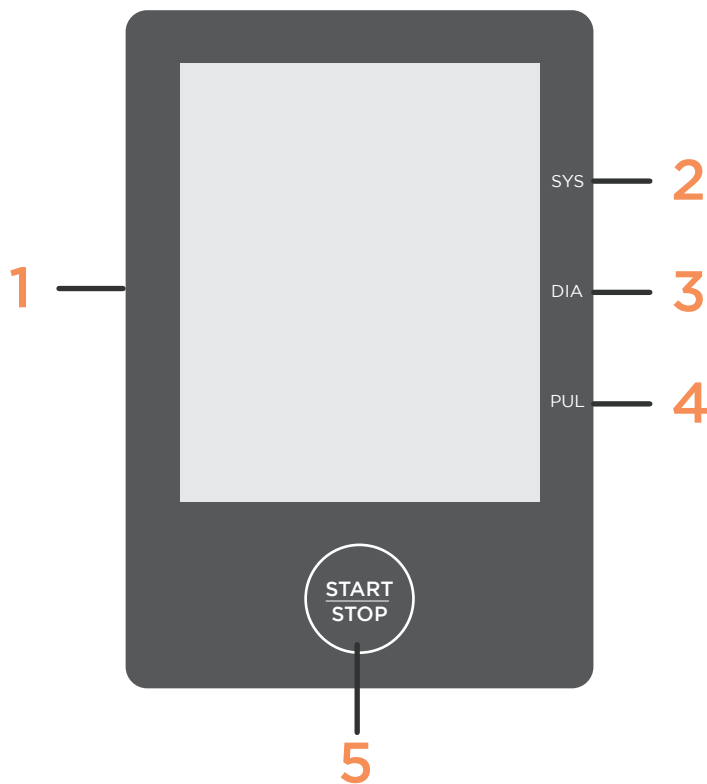

- 1 Cuff Tube Connector**
Where you will connect the cuff to your Bodytrace device
- 2 Systolic Measurement**
Displays the measurement of pressure in your blood vessels when your heart beats
- 3 Diastolic Measurement**
Displays the measurement of pressure in your blood vessels when your heart rests between beats
- 4 Pulse Measurement**
Displays your pulse
- 5 START/STOP Button**
Press to turn your device on

System Facts & Features:

- ✦ The Bodytrace device is intended for single patient use only and should not be shared
- ✦ Always make sure you take your reading on the same arm
- ✦ Only use Duracell AA batteries with the device
- ✦ Avoid caffeine, alcohol & tobacco for 30 minutes before taking a reading
- ✦ Sit down for at least 10 minutes before taking a reading
- ✦ After taking a reading, your Bodytrace will automatically transmit the measurement results
- ✦ Measurements may not be accurate if the device is used close to devices generating strong electromagnetic fields, such as microwave ovens.
- ✦ Avoid exposure of your device to direct sunlight, moisture, extreme heat, cold or humidity.
- ✦ For additional questions, call tech support at **1-800-860-4230**