


Quick Install


Bodytrace BP Monitoring System

Welcome to your new blood pressure monitoring system. If you have any questions or are in need of assistance, do not hesitate to call us at [1-800-860-4230](tel:1-800-860-4230).


- 1 Slide the battery door open on the back of the device and install the batteries provided. Close the battery door when finished.

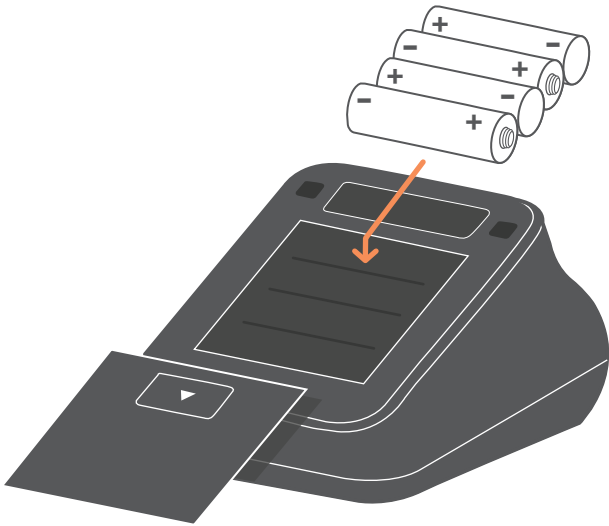

- 2 Press the START/STOP button on your device to turn it on. If your device does not turn on, check to make sure the batteries were inserted properly.

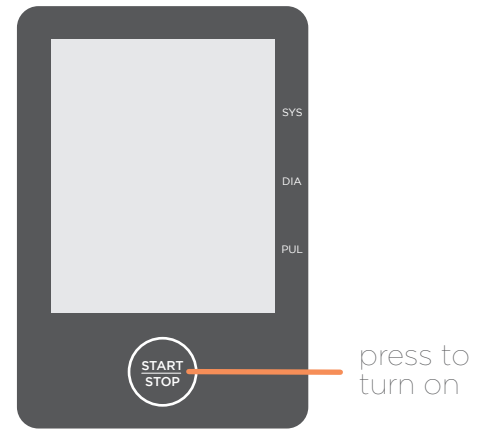

- 3 Connect the tubing from the BP cuff to the BP meter as shown. Remember to sit in a relaxed position for at least 10 minutes before taking a reading.

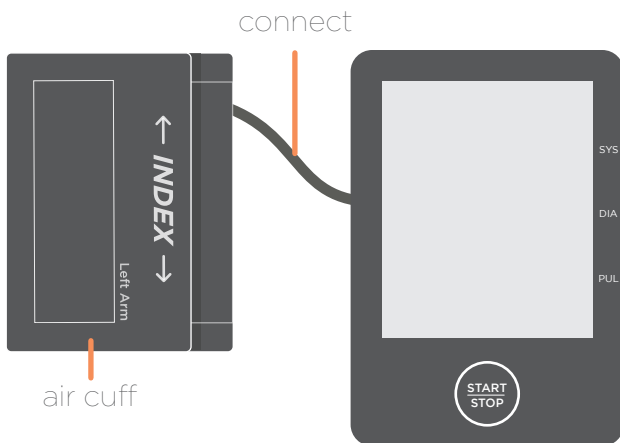

- 4 Your Bodytrace BP monitoring system is now ready to go! Please refer to the "Taking a Reading" sheet to see how to properly use your Bodytrace device.

