



Cell Pod (if applicable)

- 1 Unpack the Cell Pod equipment. (shown below)

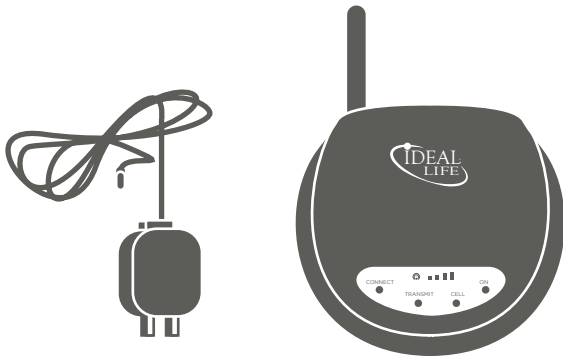


Power Cord

- 2 Plug the power cable into the back of the Cell Pod. Then plug the other end into a power outlet.

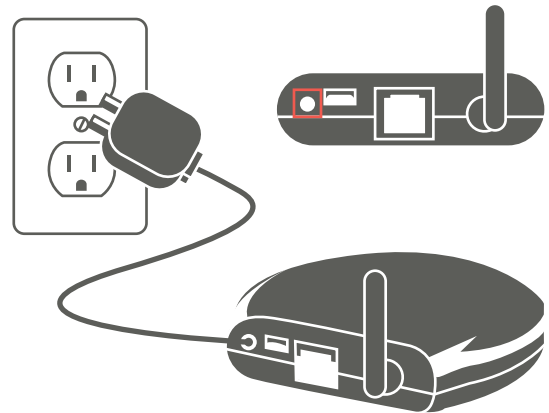

- 3 Once you have set up all units applicable to you, please call our Vitals Monitoring Team at [1-888-875-0976](tel:1-888-875-0976) then press "1" to make sure your readings were received.

Cell Pod Lights

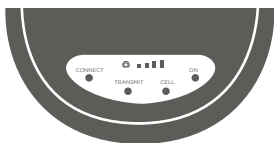

Bars

Lights represent signal strength. If none of the bars are lit this means there is little or no cell reception. Please move Cell Pod to another location to receive a stronger signal.

CONNECT

Red light will turn on when connection is reached.

CELL

Red light will turn on when cellular connection is reached during transmission.

TRANSMIT

Red light will turn on when sending reading.

ON

Red light will remain on at all times.