






Quick Install


Ellipta Propeller Vitals Monitoring

Welcome to your new Propeller sensor attachment. This guide is for the **2NET Communicator** (signal bars and house display). For 2NET2 Communicator (cell tower and house display) instructions, please flip the page over. If you have any questions or in need of assistance, do not hesitate to call us at [1-800-860-4230](tel:1-800-860-4230).

- 1 Unpack the enclosed 2Net Communicator and plug into a power outlet. Wait until the lights on the Communicator indicate the hub is ready to transmit readings.

2Net Communicator		
		Ready to receive readings from your medical device(s)
		Receiving readings from your medical device(s)
		Sending readings from your medical device(s)
		Resetting. If this is continual, please call the Vitals Monitoring Team at 1-888-875-0976

- 2 Clip the sensor on top of the inhaler. **It is critical that the sensors are attached to the correct medications.**

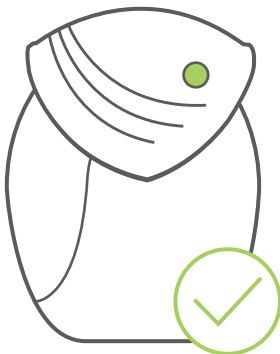

- 3 Take your medication as prescribed with the sensor on. After inhaler use the lights on your Communicator will flash indicating your information is being sent to the Vitals Monitoring Team.

After the first use please call our Vitals Monitoring Team at [1-888-875-0976](tel:1-888-875-0976) to ensure your readings were received.










Reminder: when it's time to refill, remove the sensor and place on your new inhaler **DO NOT** discard the sensor.

Quick Install


Ellipta Propeller Vitals Monitoring

Welcome to your new Propeller sensor attachment. This guide is for the **2NET2 Communicator** (cell tower and house display). For 2NET Communicator (signal bars and house display) instructions, please flip the page over. If you have any questions or in need of assistance, do not hesitate to call us at [1-800-860-4230](tel:1-800-860-4230).

- 1 Unpack the enclosed 2Net2 Communicator and plug into a power outlet. Wait until the lights on the Communicator indicate the hub is ready to transmit readings.

2Net2 Communicator				
		Ready to receive reading		Receiving reading from your medical device(s)
		Received reading		Sending reading to the Care Center
		Sent reading to the Care Center		Cannot reach a strong signal
		Resending reading		Received invalid data. Retake reading when the lights turn green

- 2 Clip the sensor on top of the inhaler. **It is critical that the sensors are attached to the correct medications.**

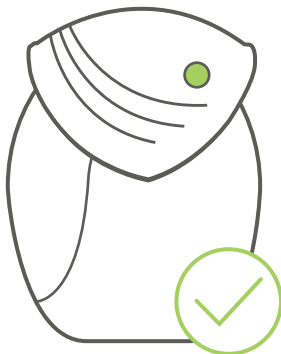

- 3 Take your medication as prescribed with the sensor on. After inhaler use the lights on your Communicator will flash indicating your information is being sent to the Vitals Monitoring Team.

After the first use please call our Vitals Monitoring Team at [1-888-875-0976](tel:1-888-875-0976) to ensure your readings were received.

Reminder: when it's time to refill, remove the sensor and place on your new inhaler **DO NOT** discard the sensor.