


VRI CareConnect

Vitals Monitoring

Enabler (if applicable)


- 1 Unpack the Enabler equipment. (shown below)

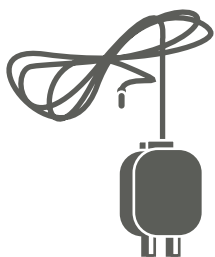

Power Cord



- 2 Plug the power cord into the back of the Enabler. Then plug the other end into a power outlet.

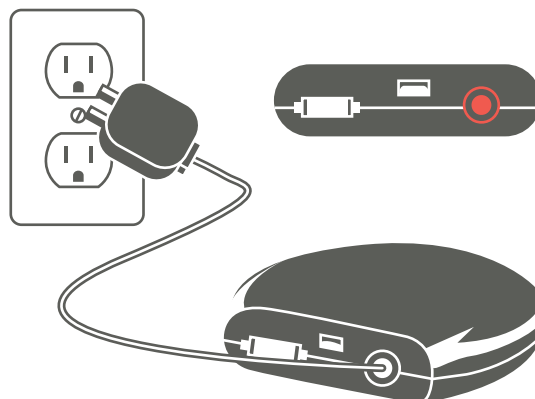

- 3 Once you have set up all units applicable to you, please call our Vitals Monitoring Team at [1-888-875-0976](tel:1-888-875-0976) then press "1" to make sure your readings were received.

Enabler Lights



Green light will blink on and off when connecting device.



Green light will remain on at all times.



Blue light will turn on when glucometer is plugged in.