


Get to Know Your MobileHelp Communicator

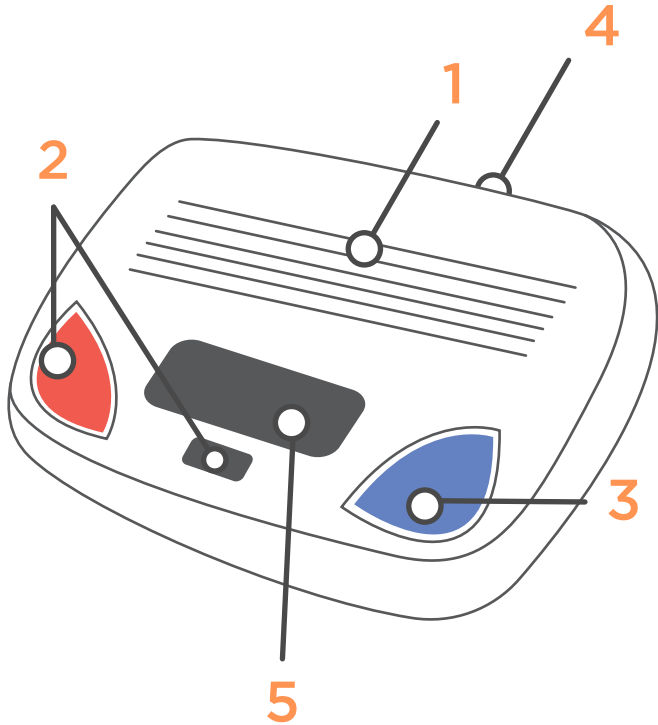

- 1 Speaker**
Will announce readings and pairings
- 2 Red Button and Test Button**
These buttons are inoperable for this device
- 3 Blue Reset Button**
Is used for pairing Bluetooth peripherals
- 4 On/Off Switch**
After you plug in the device, turn the on/off switch on to power your Communicator
- 5 Status Screen**
The unit will automatically sync when turned on
Displays battery life, signal strength, volume control, temperature, date, time, and paired devices

Voice Announcements

“System Ready”

MobileHelp Communicator on/off switch has to be turned on. Please make sure to plug the unit into a working household power outlet.

“Power not detected” or “Power detected”

Indicate either a loss of household power, or a restoral of household power. Both the red and blue buttons will be flashing. Please verify connection.

“Low Battery”

Unit is unplugged, or not connected to a working household power outlet.

“Pairing” and “Reading Announcements”

Communicator will announce what is pairing during the pairing process. While taking reading the Communicator will announce the readings it captures.

Communicator Facts & Features:

◆ Back up battery lasts up to 24 hours

◆ For additional questions, call tech support at
1-800-860-4230