


Meet Your New PERS

MXD Emergency Response System

MXD was created to provide you comfort, reassurance and peace of mind by making it possible for help to be just a touch of a button away. The MXD was designed to be easy to install and maintain.

Get to Know Your Unit

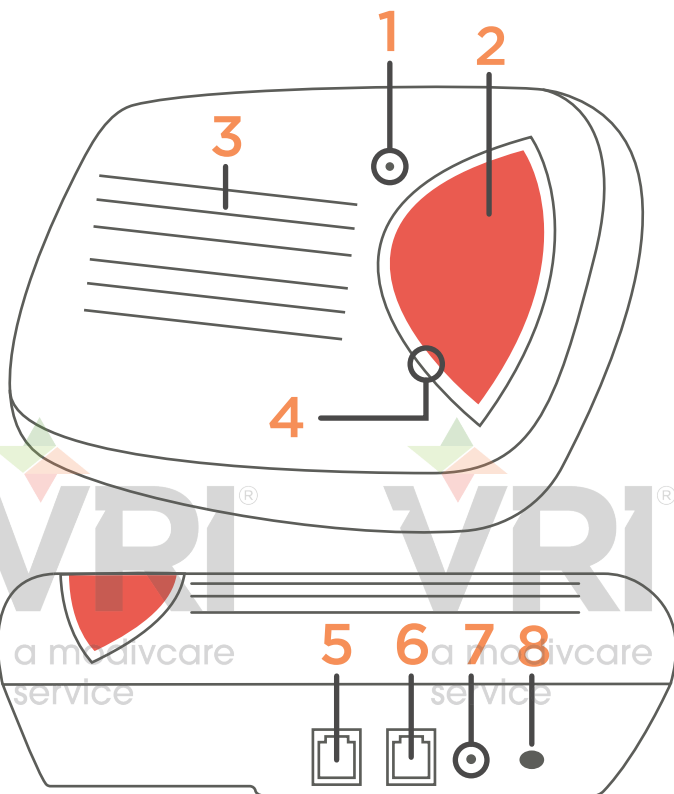

- 1 Microphone**
Used to talk to the Care Center
- 2 Help Button**
Press to call the Care Center (works just like the button)
- 3 Speaker**
Used to hear the Care Center
- 4 Status Light**
Displays the current operating condition
- 5 Wall Jack**
Used to connect the unit to your home phone service
- 6 Phone Jack**
For service provider use only
- 7 Reset Button**
Used to Reset the unit
- 8 T/L Button**
The Test and Learn Button. Used when programming and testing the unit

Unit Facts & Features:

- ✦ The button can be worn around your neck or as a wristband
 - ✦ 60+ hours of backup battery in case of AC power loss
 - ✦ The button has a range of up to 600 feet and an average 3-5 year battery life
 - ✦ Optional fall detection pendant can be added
 - ✦ Compatible with traditional phone lines, cable phones or VoIP systems.
 - ✦ Optional Medication monitoring can be added
 - ✦ Care Center monitors unit's battery status
 - ✦ Water-resistant button
- ✦ Status Light Colors:**
- **Solid Green** - Unit is properly connected
 - **Flashing Green** - AC power is not available. The unit is running on backup battery
 - **Steady Red** - Unit is trying to call the Care Center
 - **Flashing Red** - Unit is not connected to telephone service
 - **Flashing Orange** - Unit is operating on backup battery and is not connecting to working telephone service

For additional questions, call tech support at 1-800-860-4230